




# QUICK START GUIDE


Congratulations on your new ScripTalk Station 2! This Quick Start Guide will help you get started on your way to safety, privacy and independence in managing your prescription medications

## What's In the Box?

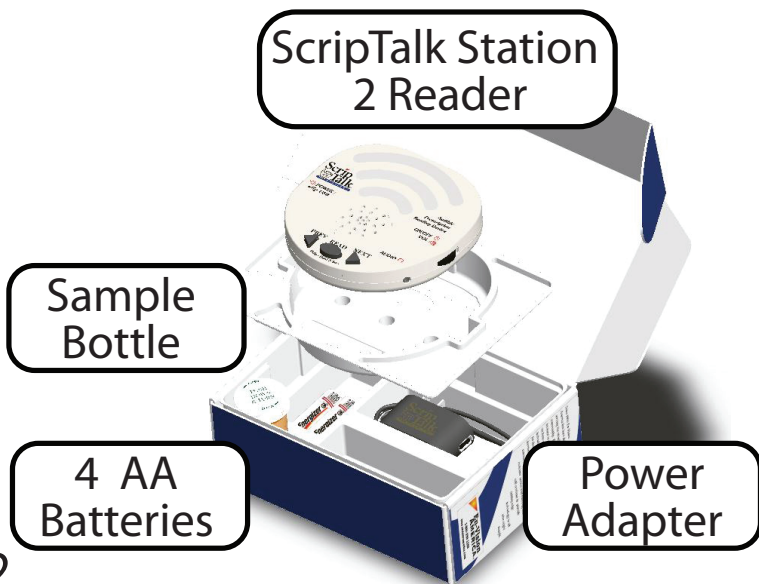



# The Buttons

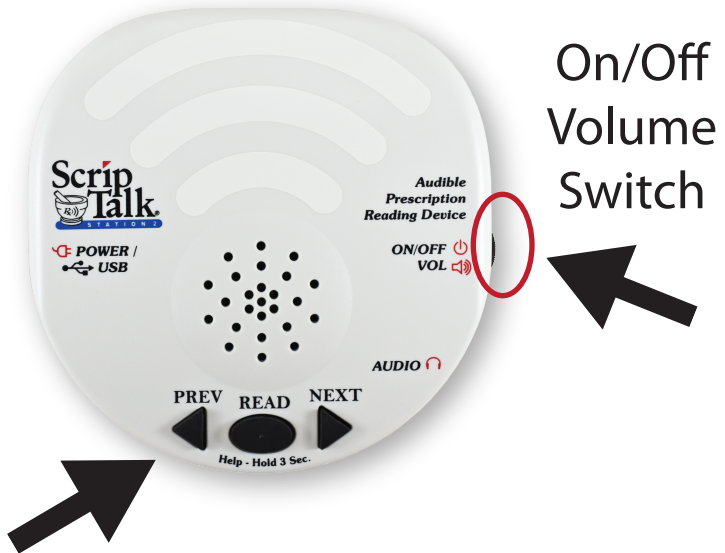

There is an Off/Off Volume switch and three buttons that will allow you to manipulate the reader and access the information on your prescription labels.

# About the Electronic Label

The talking label may be found:



- on the bottom of your bottle or container
- on the side of your bottle or box



Due to packaging or medicine content, your label may be attached as an annex tag. To access the electronic label, lift it away from the package when scanning.



# Powering the Unit

Connect the USB cable adaptor to the port on the left side of the reader. Then plug the adapter into a power source.



You can also use 4 AA alkaline batteries to power your unit. The battery access door is on the back of the unit.

If you are using batteries and forget to turn the unit off, it will beep every couple of minutes to remind you to turn it off.



# Power and Volume Control

Turn the thumbwheel switch down to power on the unit.



To adjust the volume, turn the thumbwheel switch up or down.



To shut off the unit, turn the thumbwheel switch up until you hear a click.



# On-Board Help

To listen to the instructions on how to use the unit, press and hold READ for about three seconds. To stop the instructions, press READ again.

Three  
Seconds



Press Next or Previous to move through the different topics.



# Reading a Prescription Label

- To scan, press READ. You will hear beeping.
- Place your label on the top of the reader.
- Listen to the label information.
- To stop the speaking, press READ.




# Scrolling Through the Information



While the unit is speaking the label information, you can press NEXT or PREVIOUS to move through the information or to repeat an item.

# Troubleshooting

Place your prescriptions on the reader individually. The unit can only read one label at a time.

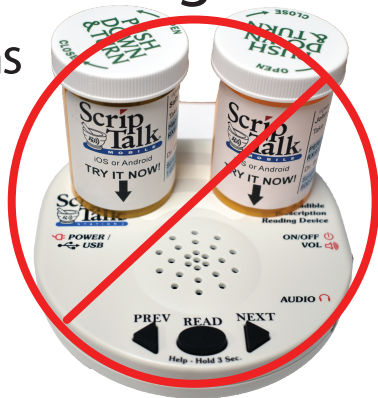

If the unit cannot find or read a label, it will give you a warning. Have a sighted friend or a family member read the label for you, and talk to your pharmacist to see about a new label.


Warning! Unable to Read Label.





# Changing the Voice Speed

To change the voice speed, press and hold NEXT or PREVIOUS for three seconds, then press NEXT or PREVIOUS to find the speed you like. Press READ to set the speed.

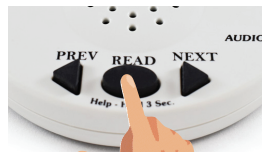

3 Seconds

Select

Save

# Spell Mode

You can have the information spelled out if you don't understand a word. Select the slowest setting in voice speed. It will say "Spell Mode." Press READ to select, then scan your label. Reset voice speed to exit.



# Accessories

A handy travel bag is available for your convenience. The insulated zippered bag is designed to hold your reader as well as your medications. And it's perfect for trips or organization of your medications.



ScripTalk Travel Case  
Headphones

\$24.99  
\$15.99

For more information or assistance,  
please contact us at:

En-Vision America  
825 4th St W  
Palmetto, FL 34221

Toll Free: (800) 890-1180  
Fax (309) 452-3643

[frontdesk@envisionamerica.com](mailto:frontdesk@envisionamerica.com)  
[www.envisionamerica.com](http://www.envisionamerica.com)



**En-Vision  
AMERICA®**